


Reference Card

For Operational Qualification (OQ) and daily testing of your sealing device

Seal Check med
Seal Check dent
Seal Check HDPE
Seal Check PP




In accordance with EN ISO 11607-2 the sealing seam must show various quality properties (5.2.2. b) with the Operational Qualification (OQ).

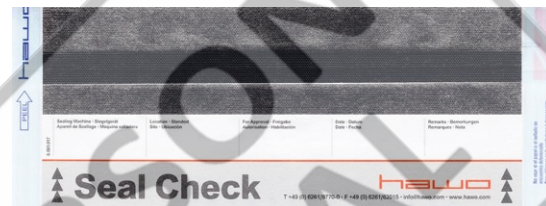


1. Intact seal for a specified seal width in accordance with EN ISO 11607-2

Following faults (images 2 to 5) may appear during a seal. In this case, the quality properties (5.3.2 b) for a sealing seam, which have to be given in accordance with EN ISO 11607-2, can not be fulfilled anymore.


Seal Check indicators are not suitable for packaging with gusseted material. We recommend the hawo InkTest.

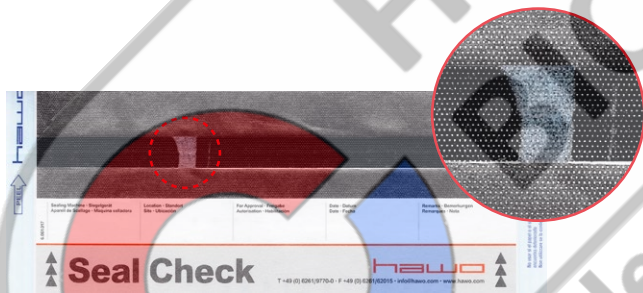


2. Channels or open seals

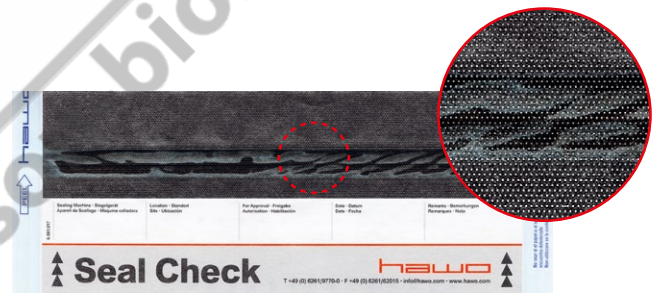

4. Material delamination or separation / temperature or contact pressure too low

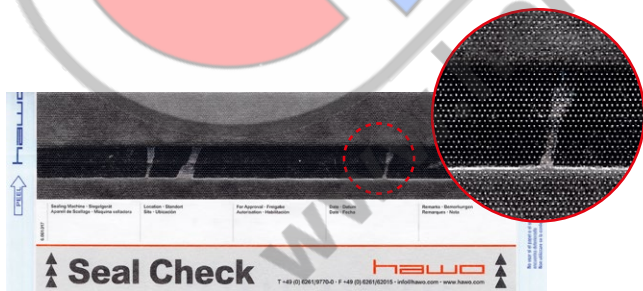

3. Punctures or tears

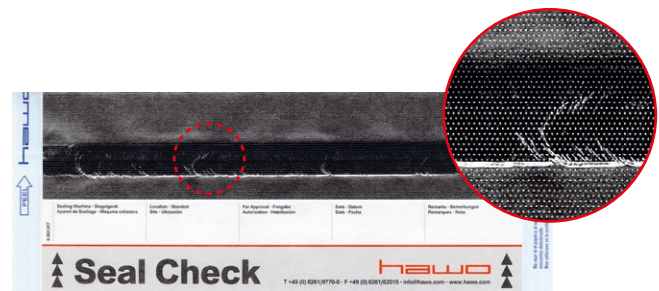

5. Temperature or contact pressure too high



LABORATORY EQUIPMENT MAINTENANCE, REPAIR, CALIBRATION AND SALES

Established in 1987, Henderson Biomedical is the UK's leading laboratory equipment sales and service provider. Our knowledgeable team can provide you with excellent sales advice on a range of different types of laboratory equipment including centrifuges, refrigerators, freezers and heat sealers.

Henderson Biomedical is also able to provide you with first class after-sales service and calibration of your laboratory equipment. We are an **ISO 17025 (UKAS) accredited calibration test laboratory** and our team of Field Service Engineers cover the whole of the United Kingdom.

Please contact us for more information on the types of equipment we supply and the different after-sales services we can offer.

Henderson Biomedical

Unit 3, Swan Close
Croydon CR0 2DZ
United Kingdom

Tel: 020 8663 4610

For sales enquiries: sales@henderson-biomedical.co.uk
For all other enquiries: info@henderson-biomedical.co.uk
www.henderson-biomedical.co.uk

